

Instructions for MacPractice Integration

MouthWatch cameras easily integrate with MacPractice version 5.0 or newer imaging software. Use the MouthWatch camera to capture images of the patient and store them directly into the patient record.


If you have an older version of MacPractice, the video will display, but button capture is not enabled.

Step 1 – Plug in the MouthWatch Camera

Make sure that you plug the USB connector into a powered USB port. Sometimes, these are located on the front or back of the computer. If your monitor has USB ports, these may not be powered.

Step 2 – Select the USB Video Device

From the main screen, hover your mouse and right click on the camera name “USB Video Device”

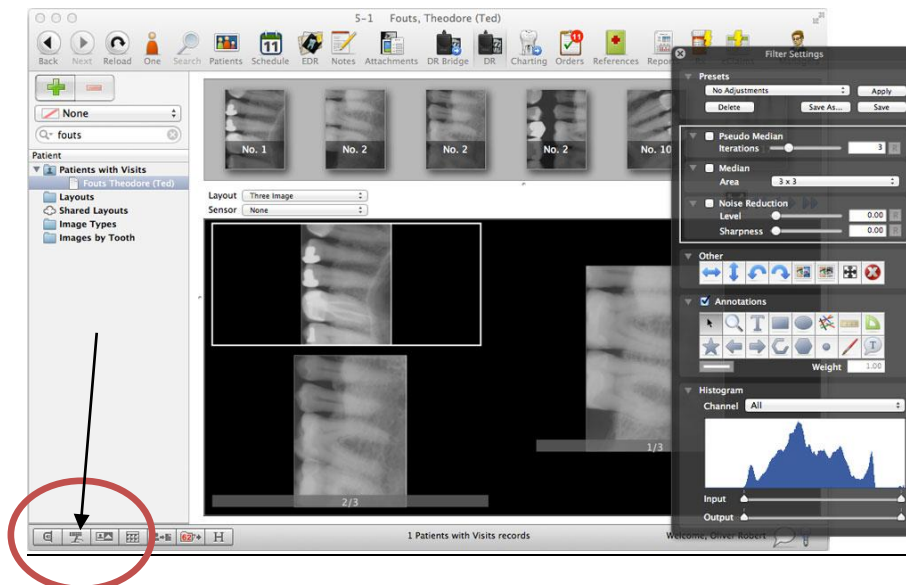

- A live video feed will stream into the image window
- Clicking the capture button on the camera will save a still photo to the layout at right
- When you close out, the images will be saved to the patient record
- All done!