



How to Clean Your MouthWatch Intraoral Camera

To ensure the longevity and optimal performance of your MouthWatch Intraoral Camera or MouthWatch Plus+ HD Intraoral Camera, it's crucial to maintain proper care.

DO's

Use OptiClear™ Intraoral Camera Sleeves with every patient for the best image quality and infection control.

To clean the camera, squeeze excess moisture from an isopropyl alcohol wipe and gently wipe the camera and cable, avoiding excessive moisture on the lens.

DON'Ts

Do not clean the camera with products that include trichloroethylene, dichloroethylene, aromatic and chloride hydrocarbons, methylene chloride, or cetones, such as degreasers, solvent cleaners, adhesive and spot removers, and metal cleaners and polishes.

Do not spray air or water directly on the lens/ light assembly.

Do not autoclave or submerge the camera.

Do not use disinfectant chemical wipes such as CaviWipes or Optim Wipes as they may damage the lens.

Do not wipe the lens excessively as this may result in fogging.

